

How to wear a face covering

A face covering should:

- Cover your nose and mouth while allowing you to breathe comfortably
- Fit comfortably but securely against the side of the face
- Be secured to the head with ties or ear loops
- Be made of a material that you find to be comfortable and breathable, such as cotton
- Unless disposable, it should be able to be washed with other items of laundry according to fabric washing instructions and dried without causing the face covering to be damaged
- Face covering should have 2 or more layers (the World Health Organization recommends 3, depending on the fabric used). They are intended to protect others and not the wearer by trapping droplets that are released when the wearer talks, coughs or sneezes. These are acceptable for use as a face covering on Hanson sites.

When wearing a face covering you should:

- Wash your hands thoroughly with soap and water for 20 seconds or use hand sanitiser before putting a face covering on
- Avoid wearing on your neck or forehead
- Avoid touching the part of the face covering in contact with your mouth and nose, as it could be contaminated with the virus
- Change the face covering if it becomes damp or if you've touched it
- Avoid taking it off and putting it back on a lot in quick succession (for example, when leaving and entering shops on a high street)

When removing a face covering:

- Wash your hands thoroughly with soap and water for 20 seconds or use hand sanitiser before removing
- Only handle the straps, ties or clips
- Do not give it to someone else to use
- If single-use, dispose of it carefully in a residual waste bin and do not recycle
- If reusable, wash it in line with manufacturer's instructions at the highest temperature appropriate for the fabric
- Wash your hands thoroughly with soap and water for 20 seconds or use hand sanitiser once removed



STAY SAFE & PROTECT YOURSELF FROM COVID-19